



**Microdermabrasion** is an effective, modern method of exfoliation, whereby dead skin cells and excess hornification is removed gently with crystals. Using the vacuum function, crystals and dead skin cells are captured hygienically and the tissue is additionally stimulated. In this way, the cell development is stimulated – the skin appears more clear, more fresh and younger. Microderm abrasion is more than just exfoliation, since physiological processes within the skin are supported. Through the stimulation of the fibroblast formation, the skin is firmed and wrinkles reduced. The skin is optimally prepared to absorb active agents after the abrasion.

## Successful treat with Microdermabrasion

Abrasion is successfully used in:

- Blemished skin
- Hyper-hornification
- Large-pored skin
- Crinkles / expression lines
- Light-induced skin aging
- Hyperpigmentation
- Superficial scars
- Stretch marks

The treatment success is demonstrated by:

- Improvement of the skin texture, skin looks even and fine-pored
- Stimulation of cell formation
- Alleviation of wrinkles and scars reliefs



**Silver Sensitive Care Concept.** The skin needs special care before and after the microderm abrasion treatment. With the Silver Sensitive products, we offer you a special skin care concept suitable during microderm abrasion treatments. The patented formula with hemp oil loaded nano-particles with micro silver was developed especially for sensitive, allergy-prone, barrier-damaged skin. The compounds are very well suited to form the natural protective function of the epidermis and to nourish it long-term. The especially high portion of micro silver, in combination with highly active, biological fermentation extracts, supports the restructuring of the skin and reduces their stimulus sensitivity.



### Silversensitive Care Concept

Free of preservatives, fragrances, dyes and parabens.

#### Silversensitive Emulsion

Nourishes the skin after an abrasion treatment and is especially suited for home care during a spa treatment.

Item No. 9135 Silversensitive emulsion, 150 ml

#### Silversensitive Enzyme Exfoliator

This is an enzyme exfoliator especially developed for the abrasion that does not have to be kept moist.

Item No. 9136 Silversensitive Enzyme Exfoliator, 50 ml

#### Silversensitive Cream Forte

This is a cream with a high concentration of active ingredients especially suited for use in the spa/institute and is suitable for use after a micro abrasion treatment. Can be used as a mask and day cream.

Item No. 9134 Silversensitive Creme Forte, 50 ml

### Abrasion Crystals

The crystals are comprised of aluminium oxide, which cannot be reabsorbed by the body and is therefore anti-allergic. The crystal sand is offered in two grain sizes:



#### Crystal Sand MACRO

For sensitive, overly callous skin as well as for the treatment of acne scars, and stretch marks.

8239.11 Crystal Sand MACRO 1 kg



#### Crystal Sand MICRO

For sensitive, atrophic and thin skin.

8239.20 Crystal Sand MICRO 3 kg  
8239.2 Crystal Sand MICRO 1 kg

### Disposable Eye Pads

The ideal eye protection during an abrasion treatment: Simply glue them over your customer's eyes before beginning the treatment. This way, you protect their eyes safely and without the, often unpleasant, eye protection with an elastic band. The pads can be removed easily and gently, completely without leaving an uncomfortable sticky glue feeling.

Item No. 8239.12 Disposable Eye Pads



Intense peeling using micro crystals sand by manual adjustment Abrasion standard

# IONTO-SKIN® ABRASION

Suitable for  
SYMBIOSIS CONCEPT by IONTO®



- Equipment features:**
- Manual cleaning function
  - Easy to remove old and fresh crystal container
  - Setting the vacuum level and amount of crystals on each knob according to skin type and sensitivity of the customer
  - Interchangeable glass tip, for sterilization and disinfection suitable

**Delivery includes:**  
Device, Standard handpiece, 2 glass tips; practical handpiece holder; 1 kg crystal sand MICRO, 1 kg Crystal Sand MACRO; 1 pair of disposable eye pads; safety goggles

**Technical data:**  
Electrical connection: 230 V / 50 - 60 Hz  
Power consumption: 90 W  
Vacuum: max. 0.85 bar  
Dimensions H / W / D (mm): 200/190/390  
Weight: 5 kg

## 8339.1 IONTO-SKIN® ABRASION

Item No.	Accessories
8317.8	COMBI cart W 190 x H 200 x D 390 mm for Ultrasound and IONTO-SKIN ABRASION
8339.003	Glass tip
8339.013	Safety goggles for SKIN ABRASION treatment
8239.20	MICRO crystal sand, 3 kg
8239.2	MICRO crystal sand, 1 kg
8239.11	MACRO crystal sand, 1 kg
8339.001	Air filter for container, 4 pieces Alternating rhythm about every 6 weeks.
8239.12	Disposable eye pads, pack of 50 pairs

Item No.	Accessories
9135	Silversensitive Emulsion, 150 ml
9136	Silversensitive Enzymepeeling, 50 ml
9134	Silversensitive Creme Forte, 50 ml

Intense peeling using micro crystals sand with easy handling by electronic device control

# IONTO-SKIN® ABRASION COMFORT

Suitable for  
SYMBIOSIS CONCEPT by IONTO®



- Equipment features**
- Vacuum and dosage level can be set quickly at the touch of a button.
  - Crystal flow and vacuum power can be set to a precise level between 0 and 100 %.
  - Values are shown in an easy-to-read display.
  - Preprogrammed intensity levels: sensitive, medium or intense for face and body.
  - 5 frequently used settings can be stored individually in memory.
  - The self-cleaning function can be activated effortlessly at the touch of a button.
  - Practical, integrated hand unit holder.
  - Container for used and fresh crystals is easy to remove.
  - The comfort hand unit lets you tackle skin problems more directly.
  - The glass nozzle can be replaced; suitable for sterilization and disinfection



## 8330 IONTO-SKIN® ABRASION COMFORT

Item No.	Accessories
8317.8	COMBI cart for MICROABRASION + ULTRASOUND
8339.003	Glass tip
8339.009	Glass nozzle, 1,5 mm
8339.013	Safety goggles for SKIN ABRASION treatment
8239.20	MICRO crystal sand, 3 kg
8239.2	MICRO crystal sand, 1 kg
8239.11	MACRO crystal sand, 1 kg
8339.001	Air filter for container, 4 pieces Alternating rhythm about every 6 weeks.
8239.12	Disposable eye pads, pack of 50 pairs

**Delivery includes:**  
Combi-device; Comfort hand unit; Ultrasound head for the face 2,5 cm²; 1 Glass tips, 2 pcs (sterilizable); 1 Glass nozzles 1,5 mm (sterilizable); MICRO crystal sand, 1 kg; MACRO crystal sand, 1 kg  
1 pair of disposable eye pads  
Safety goggles  
1 x SONO contact gel Aloe Vera


**Technical data:**  
Electric connection: 230 V / 50 - 60 Hz  
Power consumption: 100 Watt  
Vacuum ABRASION: max. 0.85 bar  
Dimensions H/W/D (mm): 235 / 345 / 305  
Weight: approx. 10 kg

Item No.	Accessories
9135	Silversensitive Emulsion, 150 ml
9136	Silversensitive Enzymepeeling, 50 ml
9134	Silversensitive Creme Forte, 50 ml



The perfect combination for immediate visible treatment success  
Microabrasion + Ultrasound

# IONTO-SKIN® ABRASION SONO

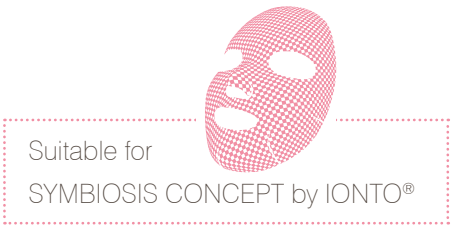

Equipment features

- ABRASION**
- Crystal flow and vacuum strength can be set manually to a level between 0 and 100%
  - Preprogrammed intensity levels: sensitive, medium or intense for face and body
  - 5 frequently used settings can be stored individually in the memory
  - The self-cleaning function can be activated effortlessly at the touch of a button
  - Container for used and fresh crystals is easy to remove
  - The glass nozzle and glass peak can be replaced; suitable for sterilization to prevent infection

- ULTRASOUND**
- 3 MHz = superficial effect and 1 MHz = depth effect
  - 21 permanently stored programs, 20 individual memories
  - The optical and acoustic coupling control indicates whether there is sufficient contact with the skin
  - Ultrasound head for face, 2.5 cm²; ultrasound head 5 cm² for body treatments



Microabrasion and ultrasound is THE effective combination in cosmetic special treatment area. By Microabrasion wrinkles / expression lines are immediately visually impaired and refines pores. Therefore, the skin is more receptive to active ingredients. Through the ultrasonic treatment, the active ingredients are anchored and deposited in the skin. The versatile applications of ultrasound, such as lifting or phonophoresis make this combination system an all-rounder for a variety of cosmetic treatment concepts.



Make use of ultrasound with 1 and 3 MHz for:

- Cell regeneration
- Increase in metabolism
- Promotion of the intercellular substance exchange
- Accelerate the lymph circulation
- Improving blood circulation
- Improved absorption of active ingredients
- Increased and sustained effect of treatment products
- Relaxation of the muscles
- Positive effect for blemished skin, rosacea and cellulite

8440	IONTO-SKIN® ABRASION SONO
Item No.	Accessories
8422.005	Ultrasound-Treatment head 2,5 cm²
8422.001	Ultrasound treatment head for body treatments, 5,0 cm²
8339.003	Glass tip
8339.009	Glass nozzle, 1,5 mm
8339.013	Safety goggles for SKIN ABRASION treatment
8239.20	MICRO crystal sand, 3 kg
8239.2	MICRO crystal sand, 1 kg
8239.11	MACRO crystal sand, 1 kg
8339.001	Air filter for container, 4 pieces Alternating rhythm about every 6 weeks.
8422.009	SONO contact gel neutral, 200 ml
8422.016	SONO contact gel Basic, 200 ml
8422.002	SONO contact gel Aloe Vera, 250 ml
8422.004	SONO contact gel ACE, 200 ml
9146	Lifting gel, 250 ml
9147	Body Contouring Gel, 250 ml
9136	Silversensitive Enzympeeling, 50 ml
9134	Silversensitive Creme Forte, 50 ml
9135	Silversensitive Emulsion, 150 ml

**Delivery includes:**  
Combi-device; Comfort hand unit; Ultrasound head for the face 2,5 cm²;  
1 Glass tips, 2 pcs (sterilizable);  
1 Glass nozzles 1,5 mm (sterilizable)  
MICRO crystal sand, 1 kg  
MACRO crystal sand, 1 kg  
1 pair of disposable eye pads  
Safety goggles  
1 x SONO contact gel Aloe Vera

**Technische Daten:**  
Electric connection: 230 V / 50-60 Hz  
Power consumption: 100 Watt  
Vacuum ABRASION: max. 0,85 bar  
Ultrasound frequency: 1 MHz und 3 MHz  
Dimensions H/W/D (mm): 345 / 400 / 235  
Weight: approx. 14 Kg