

## Short-term visible results

More than ever, this is what customers demand of cosmetic treatments. IONTO-COMED offers you an effective apparative support, which is distinguished by top efficacy paired with clearly understandable ease of use.

From the basis treatment devices to special and additional treatment facilities to innovative anti-aging concepts with immediate effects.

Trust the quality of the market leader in machine-supported cosmetics.

Basic treatment | Anti-Aging-Special treatment

# Treatment concepts



From page 52

SYMBIOSIS CONCEPT  
by IONTO®

From page 58

Machine-supported  
Basic treatments

From page 76

Machine-supported  
Special treatments

From page 100

Steamers

From page 104

Treatment systems



# SYMBIOSIS CONCEPT by IONTO®

The perfect symbiosis of apparative treatment and innovative mask compound ingredients

Versatile - Effective - Fast

The basis: TT-Cellulose active substance mask

Cellulose – The unique mask compound ingredient:

The IONTO-SYMBIOSIS concept uses a natural substance as a carrier for highly concentrated, perfectly balanced active substance concentrates: cellulose. Thanks to its unique characteristics, cellulose is perfectly suited for apparative treatments:

Optimal results thanks to perfect adaptation, like a second skin:

Cellulose has an extremely fine, gel-like structure and adapts to the surface of the skin perfectly, so that the active ingredients can be absorbed evenly and in high concentration.

Fine fibres for optimal infiltration of active substances:

The fibres used for the SYMBIOSIS CONCEPT by IONTO® are made of extremely fine cellulose and have a diameter of 1/1000 of a human hair. Thanks to the network structure, the active ingredients can be optimally transported into the skin.

Not only a carrier substance – The use of cellulose in medicine:

Cellulose is used in medicine as a moist wound dressing due to its high capacity for absorbing and delivering liquids and is highly valued in this area due to its strong healing effect.



Apply, treat, remove –  
Using the cellulose mask is so simple




Practical handling for fast work

Every mask is:

- individually packed
- completely pre-cut, incl. pads for the eyes
- pre-soaked in a high-quality active ingredient concentrate
- wrapped in a protective foil to make the application easier

Hygienic and High Purity –  
Safety for Aestheticians and Customers

The cellulose used in the SYMBIOSIS CONCEPT by IONTO® is derived from coconut and fermented in an elaborate process. The basis for the later masks is formed, which is cut using laser technology and packaged according to the highest hygiene standards.

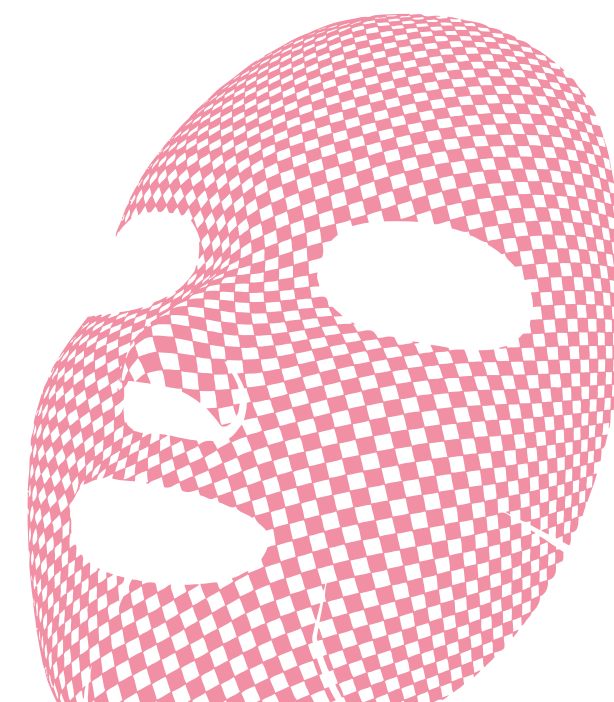



## Selected Ingredients – Immediately Visible Treatment Results

The TT Cellulose Masks are optimally adjusted to various skin types. There are 30 ml of active ingredient concentrate in each mask, which replaces ampoules. Simply choose the appropriate mask for your customers:

### Sensitive Skin:

#### Cherry Blossom

Thanks to carefully chosen ingredients, the Cherry Blossom Mask relieves redness, relaxes stressed skin and gives it moisture.



### Problem Skin:

#### Green Tea

The Green Tea Mask has antioxidant, detoxifying and calming effects. It combines the anti-inflammatory characteristics of green tea with the wound healing power of camomile.



### Dry and Tired Skin:

#### Deep Sea

The moisturizing Deep Sea Mask contains SEAWEED EXTRACT and DEEP SEA WATER rich in minerals, which revives and cools the skin.



### Moisture Booster:

#### Hyaluron

The Hyaluron Mask is the ultimate moisture booster for dry skin. Selected ingredients provide an intense supply of moisture and ideal nourishment. High-dosed HYALURON increases the skin's water binding ability and gives it a fresh and firm look.



### First Signs of Skin Aging:

#### Collagen

The luxurious Collagen Mask works against the first signs of skin aging. Its regenerating active ingredients, such as high-dosed COLLAGEN, lets skin that is developing the first lines, appear smooth.



### Mature Skin:

#### Caviar Q10

The Caviar Q10 Mask helps reduce the visible signs of aging, and retains elasticity giving the skin a healthy, younger complexion. In addition to COENZYME Q10, which is an essential source of energy for the skin cells, the Caviar Q10 Mask contains SODIUM ASCORBYL PHOSPHATE, which impedes melanin synthesis.



### Sensational effects for the eyes:

#### Cellulose Eye Mask

The special revitalizing Cellulose Eye Mask formula supports the reduction of fine lines and helps fade swelling around the eyes. The pads work gently, without irritating sensitive skin. They provide intense moisture penetration, firms the skin around the eyes and increases elasticity. The main active ingredients of the Cellulose Eye Mask are HYALURON and ALGAE EXTRACT.



## Optimal Use of Cellulose Masks

### The Symbiosis of masks and machine-supported treatment

- 1 Machine-supported work with the cellulose-masks**  
Infiltrates concentrated active substances of the mask and use it simultaneously as a coupling medium.
- 2 Application of the cellulose mask after a machine-supported treatment -**  
Use the multi-functionality of the Cellulose Mask: After an apparative treatment, it acts not only as a high-dosed active substances mask, but also as a wound healing dressing, which stops the excessive trans-epidermal water loss for example, after abrasive treatments.
- 3 Manual application of the Cellulose-Mask**  
Mask and massage in one – save treatment time and treat your customers to an extraordinarily pleasant massage experience.



## Optimise the SYMBIOSIS CONCEPT by IONTO®: The appropriate Homecare products

Especially suitable for use at home, we offer TT active substances masks based on the carrier material SILK. These masks contain the same active substances complexes as the Proline Cellulose Masks, however the carrier substance is different. TT Silk Masks are meant to be sold to your customers for use at home. In this way, the successful treatment used in the spa according to the IONTO SYMBIOSIS Concept can be supported sustainably and maintained at a higher level until the next treatment. The high quality TT Silk active ingredients masks are only available to customers of IONTO partner institutes EXCLUSIVELY.



# Treatment concepts

## Skin analysis



Basic treatments .....  
Special treatments —

## Cosmetics Basic treatments



## Radio frequency



## Medium-frequent applications



## Meso + Systeme



## Microdermabrasion



## Ultraschall



## Steamers



## Treatment systems





# Skin analysis 2.0

High-tech in a compact format



The epidermis is the focus of almost all dermo-cosmetic treatments. A professional skin analysis is necessary in order to achieve optimum results. The better moisture and grease content, pores and wrinkle structure, pigmentation, infiltrates etc. are recognised, the easier it is to optimise the treatment concept in order to achieve maximum results. The technical skin analysis using optical measurement increases the competence and safety of the case history considerably.

- Properties**
- Menu guidance via touchscreen display is possible
  - Integrated measuring sensors in order to determine skin hydration
  - Immediate evaluation of the images by software in the camera
  - Comparison of images with stored images as a guideline
  - Option of creating a customer file and saving the measuring results
  - Graphics overall skin assessment
  - Internet access
  - Data transfer via Bluetooth and USB
  - Screen transfer via WiFi and HDMI
  - Photographic camera with image memory

- Scope of delivery**
- Charging station
  - 2 battery packs
  - USB cable
  - 2 boxes containing sebum paper (each with 50 pieces)
  - Pen

- Technical data:**
- |               |                                     |
|---------------|-------------------------------------|
| Camera:       | Dual 5 Megapixel                    |
| Illumination: | Normal, polarisation, UV            |
| Touchscreen:  | 10.16 cm Grass Multi C-Touch Screen |
| LCD:          | TFT LCD WVGA                        |
| Wireless:     | WiFi + Bluetooth                    |
| Battery pack: | Exchangeable 1150mAh/Li-ION, 3.7V   |
| Software:     | Android Gingerbread 2.3             |

8436.1 IONTO-SCOPE APM 100

# IONTO-SCOPE APS II 100

Skin analysis camera with measuring of:

- Skin hydration
- Elasticity
- Sebum deposits
- Pore size
- Melanin content
- Sensitivity
- Wrinkles
- Skin problems

As easy, intuitive and as practical to use as a Smartphone, the new IONTO-SCOPE skin analysis devices offer maximum performance and a considerable amount of application comfort: pin sharp images are almost automatically converted into understandable and easily comprehended graphics with assessment data and shown on the display or a connected tablet or monitor. This allows you and your clients to view the images together and to discuss the resulting treatment recommendations.

- Properties**
- Menu guidance via touchscreen display is possible
  - Integrated measuring sensors in order to determine skin hydration
  - Immediate evaluation of the images by software in the camera
  - Option of creating a customer file and saving the measuring results
  - Graphics overall skin assessment
  - Internet access
  - Camera can be connected to a screen via USB, Bluetooth or WiFi
  - Photographic camera with image memory

- Scope of delivery**
- Charging station
  - 2 battery packs
  - USB cable
  - 2 boxes containing sebum paper (each with 50 pieces)
  - Pen

- Technical data:**
- |               |                                   |
|---------------|-----------------------------------|
| Camera:       | 5 Megapixels                      |
| Illumination: | Normal, polarisation, UV          |
| Touchscreen:  | 5.59 cm Grass R-Touch Screen      |
| LCD:          | TFT LCD QVGA                      |
| Wireless:     | Bluetooth                         |
| HDMI:         | HD1080p                           |
| Battery pack: | Exchangeable 1150mAh/Li-ION, 3.7V |
| Software:     | Android Gingerbread 2.3           |

8436.2 IONTO-SCOPE APS II 100



# IONTO-SCOPE APM 100

Skin analysis camera with measuring of:

- Skin hydration
- Elasticity
- Sebum deposits
- Pore size
- Melanin content
- Wrinkles
- Skin problems

- Properties**
- Menu guidance via touchscreen display is possible
  - Integrated measuring sensors in order to determine skin hydration
  - Immediate evaluation of the images by software in the camera
  - Comparison of images with stored images as a guideline
  - Option of creating a customer file and saving the measuring results
  - Graphics overall skin assessment
  - Internet access
  - Data transfer via Bluetooth and USB
  - Screen transfer via WiFi and HDMI
  - Photographic camera with image memory

- Scope of delivery**
- Charging station
  - 2 battery packs
  - USB cable
  - 2 boxes containing sebum paper (each with 50 pieces)
  - Pen

- Technical data:**
- |               |                                     |
|---------------|-------------------------------------|
| Camera:       | Dual 5 Megapixel                    |
| Illumination: | Normal, polarisation, UV            |
| Touchscreen:  | 10.16 cm Grass Multi C-Touch Screen |
| LCD:          | TFT LCD WVGA                        |
| Wireless:     | WiFi + Bluetooth                    |
| Battery pack: | Exchangeable 1150mAh/Li-ION, 3.7V   |
| Software:     | Android Gingerbread 2.3             |

8436.1 IONTO-SCOPE APM 100



Cleaning, deep cleansing, brush massage



Treatment device  
IONTO-PEEL®



The application of the IONTO-PEEL® device is a „must“ for preparing the skin for the following treatment.  
Dead skin cells of face and body are removed.  
The skin absorbs the active agents of the caring preparations much better.  
Improves the effect of the apparative peeling by the use of cosmetical peeling preparations.



Sponge attachments or goat-hair brush for precleaning, horse-hair brushes for body treatments or for robust body skin. For a deep cleansing perform wet grinding with sponge attachment or dry grinding with a grindstone .



**Equipment features**  
Light-weight handpiece  
Proven technology  
Constant rpm - even when pressure is applied to the brush  
Infinitely adjustable  
Extensive line of accessories

**Included in delivery:**  
Device and peel handpiece (Item No. 60139).  
Accessories must be ordered seperately.

**Technical data:**  
Electrical connection: 230 V / 50 - 60 Hz  
Power: 13 VA  
Dimensions H/W/D (cm): 7 / 19 / 19  
Weight: approx. 1,5 kg  
Speed: 20 bis 600 revs/min

**8815.8312 IONTO-PEEL® with holder for hand unit**

Item No.	Accessories and spare parts
8365	9-piece accessory set for IONTO-PEEL®
Components	

8265	Cylindrical goat-hair brush
8361	Sponge holder
8257	Goat-hair brush, Ø 45 mm, for the face
8260	Grindstone, Ø 45 mm, for face and body
8357	Horse-hair brush, Ø 45 mm, for robust body skin
8258	Goat-hair brush, Ø 20 mm, for the face
8358	Horse-hair brush, Ø 20 mm, for robust body skin
8360	Grindstone, Ø 20 mm,
8361.001	Replacement sponges, 1 pack of 10 pieces
8265	Goat-hair cylinder brush, 9 cm high, the „big one“ for the neck and cleavage

Everything in the right place. Practical accessory tray with matching recesses for all the accessories of the accessory kit, including use of the handpiece of IONTO-PEEL® device. Accessories to be ordered separately.

Item No.	Accessories
8266	Horse-hair brush-cylinder, 12 cm high for body treatments
8259	Body plate brush, 60 mm
8800.5	Accessory shelf
9132	Biozym peeling 250 ml

Cleansing and vacuum massage



Treatment device  
IONTO-VAC® SPRAY



The suction effect of IONTO-VAC® SPRAY loosens tallow and low-lying infiltrates , transports them to the surface, where they can be removed easily.  
The produced low underpressure can be used also for the vacuum massage with the help of special accessories.  
Blood circulation and lymph flow are stimulated when a local suction is created on the skin. After treatment, soothe or refresh the skin simply by using the spray.



Equipment features:

- High vacuum performance
- Double function svacuum and spray
- Multiple use due to extensive line of accessories

Included in delivery:

IONTO-VAC® SPRAY (Item No. 8815.8513 and 8825.8513 respectively) and 2 tubes and filter papers 10 pieces (Item No. 8362.002).  
Accessories must be ordered separately.

Technical data:

Electrical connection: 230 V / 50 - 60 Hz  
Power: 13 VA  
Dimensions H / W / D (cm): 7 / 19 / 19  
Weight: approx. 1,5 kg  
Vacuum: - 0,5 bar  
Pressure under continuous operation: 2,0 bar

8815.8513	IONTO-VAC® SPRAY with holder for hand unit
Item No.	Accessories and spare parts
32721	Tube, 1,20 m long
8362.002	Filter papers, 1 pack à 10 pieces for adapter
9223.91	Anti-Aging ampule, 10 pieces à 2 ml
9223.92	Hydro-ampule, 10 pieces à 2 ml
9121.6	Almond oil 100 ml
9121.7	Anti-cellulite-oil 100 ml

8366	Acessoy set for IONTO-VAC® SPRAY, 7-pieces
8356	Glass ventouse, small, for removing skin impurities
8256	Glass ventouse, large, for removing larger infiltrates
8456	Glass ventouse, oval, for vacuum massage (for wrinkles)
8264	Vacuum bell, Ø 25 mm, for massage in the neck and cleavage area
8263	Vacuum bell, Ø 45 mm, for massage on the body and back of the neck
8262	Spray bottle, for nebulizing lotions
8362	Adapter, holder for glass ventouses and suction cups



Calm and activate  
Vascular training



The heating and cooling function offers a wide application spectrum: Heat can significantly enhance the effect of your products' ingredients. Both heat and cold stimulate metabolism. When only cold is applied, vessels react by briefly narrowing and then intensifying the flow of blood a natural vascular workout.

The IONTO ICE & HEAT wand offers a unique means of alternating heat and cold treatments, because the wand switches directly from heat to cold and vice versa without pausing. This stimulates the metabolism and allows the skin to absorb active agents more readily.



Treatment device  
IONTO-ICE & HEAT

Suitable for  
SYMBIOSIS CONCEPT by IONTO®



- Equipment features:**
- Cold treatment from +5 to +15 °C
  - Heat treatment from +20 to +38 °C
  - Safety shut-down after 30 min. continuous operation
  - Cold and heat treatment with one handpiece

**Included in delivery:**  
Device and ICE&HEAT wand

**Technical data:**  
Modular SL  
Electrical connection:: 230 V / 50 - 60 Hz  
Power: 30 VA  
Dimensions H/W/D (cm): 7 / 19 / 19  
Weight: approx. 1,6 kg  
Max. output voltage: max. 4 V

8815.8281 IONTO-ICE & HEAT with holder for hand unit	
Item No.	Accessories and spare parts
50222	Replacement ring ICE & HEAT
9223.91	Anti-Aging ampoules, 10 pieces à 2 ml For incorporation during heat treatment.
9223.92	Hydro ampoules, 10 pieces à 2 ml For incorporation during cold treatment.



Disinfecting blemished skin  
 Activating and vitalizing of poor circulation, sensitive skin



A HF treatment promotes blood circulation of the skin and has a disinfecting effect. During treatment, the HF wand is placed on the affected area immediately after removing skin impurities; the produced ozone has an optimum disinfecting effect and prevents inflammations. Moreover, it is possible to introduce active agents improvingly into the skin. Simply apply skin care products (without alcohol) and slide over the skin.



Treatment device  
 IONTO-HF



- Equipment features:**
- Low-noise working
  - Easy-to-use, secure coupling mechanism for electrodes
  - Different electrodes for versatile use
- Included in delivery:**  
 Device and HF wand, please order electrodes separately.

**Technical data:**  
 Electrical connection: 230 V / 50 - 60 Hz  
 Power: 10 VA  
 Dimensions H / W / D (cm): 7 / 19 / 19  
 Weight: approx. 1,2 kg  
 Secondary voltage: max. 45 kV

8373.1	IONTO-HF
Item No.	Accessories and spare parts
8273.002	HF roller electrode
8273.004	HF electrode, open
8273.003	HF comb electrode
8273.001	HF electrode, closed
8800.7	HF wand holder, 1 piece



Iontophoresis for introducing active agents  
Improvement of cell metabolism deep cleansing by desincrustation



Treatment device  
**IONTO-GALVANO®**

**i** IONTO-GALVANO® is also suitable for galvanic footbath.



The application of a constant, galvanic direct current (iontophoresis) is based on the finding that ions migrate in a specific direction in a field with a direct current, i.e. always in the direction of the oppositely charged electrode. The IONTO-GALVANO® utilizes this effect in order to take agents that dissociate into ions and drive them into the deeper layers of the skin. The treatment is used for regeneration and improvement of cell metabolism and for introducing special active agents. Moreover, deep cleansing is possible due to desincrustation as well as a galvanic foot bath.

Item No.	Accessories
1 8267	Tampon electrode & Exanche tampons
2 8267.001	Tampon electrode & Exanche tampons
3 8268	Sponge electrode
4 8361.001	Sponge electrode
5 8242	Face electrode
6 8242.1	Face electrode (2 pcs)
7 8251.5	Counter electrode, 15 x 16 cm
8 8241	Counter electrode (small)
9 8251	Surface electrode (big)
10 8247.2	Counter electrode (2 pcs) incl. sweatband
11 9162	Fleece roll
12 8228.011	Sweatband (2 pcs)
13 8800.8257	Spiral cable set, 4-fold

Recommended basic equipment:	
Item 8242.1	Face electrodes, 1 pair
Item 8251.5	Counter electrode, 1 piece, 15 x 16 cm
Item 8800.8257	Spiral cable Set, 8-fold
Item 9162	Fleece roll, 1 roll of 500 m

**Equipment features:**  
Iontophoresis device for facial and body treatments; efficiently drives active cosmetic agents into the deeper layers of the skin; 0 - 2 mA for facial treatments and 0 - 10 mA for body treatments. Selection of programs for facial and body treatments. Time setting for programming treatment duration. Electronic safety monitoring.

**Delivery includes:**  
Device without accessory. Please order accessory separately!

Technical data:	
Electrical connection:	230 V/50 - 60 Hz
Power:	13 VA
Dimensions H/W/D (cm):	7 / 19 / 19
Weight:	approx. 1,3 kg
Max. output current:	max. 10 mA



**8815.8120 IONTO-GALVANO® with holder for hand piece**

**Included in delivery:**  
Device without accessories. Accessories to be ordered separately!

Item No.	Accessories
9223.90	Super-Clean ampoules, 10 pieces à 2ml
9223.92	Hydro-ampoules, 10 pieces à 2ml
9223.91	Anti-Aging ampoules, 10 pieces à 2ml
9022	Active creme 50ml



Apparative lymphatic and micromassage  
 Promotion of microcirculation  
 For normal to blemished skin and cellulite



Treatment device  
**IONTO-SKIN® REGULATOR**

Suitable for  
 SYMBIOSIS CONCEPT. by IONTO®



IONTO-SKIN® REGULATOR Micromassage let your hands slide softly across the face or body of your clients and generate regenerative vibrations with a calming and relaxing effect on skin and muscles. The IONTO-SKIN® REGULATOR has a working range of 5 - 50 Hz, provides a deep tissue massage and is therefore used to promote blood circulation. The micromassage ensures that the tissue is provided with an optimal supply of oxygen and nutrients and that metabolic waste products are removed by an activated lymph flow. The result is an optimal balance between input and output and skin looks healthy and brilliant. Moreover, the muscles are relaxed. In combination with a collagen BIOMATRIX Mask the skin is optimally provided with moisture and active agents t and the skin looks immediately younger and fresher.

Item No.	Accessories
8228.050	Vinyl gloves, size S, 100 pcs.
8228.051	Vinyl gloves, size M, 100 pcs.
8228.052	Vinyl gloves, size L, 100 pcs.
8228.011	Sweat bands, 1 pair, one size, white
8228.007	Holder for the sensomembrane (plastic film)
8228.009	Reusable adhesive electrodes
9147	Body contouring gel, 250ml
8228.100	Spiral cable with push-button (connects adhesive electrode with unit)
8228.101	Wand incl. supply line to the device
8228.003	Disposable adhesive electrodes, 30 pcs.
8228.031	Sensomenbrane, 1 roll, 30 cm length, 30 cm width



- Equipment features:**
- Working range of 5 - 50 Hz provides a deep tissue massage.
  - Double application: Micromassage solo or BIOMATRIX application
  - Double effect: activates the flow of lymphs and increases effectiveness of skin care products.

**Delivery includes:**  
 Device, wand, vinyl gloves, size S (Item No. 8228.050), adhesive electrodes, 4 pcs. (Item No. 8228.009), sweat bands (Item No.- 8228.011), cable with push button (Item No. 8228.013), wand incl. instruments feed (Item No. 50314), powder (Item No. 8228.005)

**Technical data:**  
 Electrical connection: 230 V / 50 - 60 Hz  
 Power: 13 VA  
 Dimensions H/W/D (cm): 7 / 19 / 19  
 Weight: approx. 1,5 kg  
 Max. output voltage: max. 600 Vss  
 Frequency: 5-50 Hz

**8815.8228 IONTO-SKIN® REGULATOR**

**8409.58 BIOMATRIX concept Starter package**





Effective treatment using non-woven substances and micromassages



# BIOMATRIX- Concept



The unique BIOMATRIX Concept is the interaction between BIOMATRIX Masks, micro massages and your hands. The treatment combines wellness and beauty to create a completely extraordinary wellness experience. The BIOMATRIX Masks are made out of collagen, which has an exceptional moisturising effect. The masks are available in four different active agents combinations.

During the treatment, the customer experiences pleasant, gentle vibrations. The facial skin and muscles relax and mimic wrinkles are reduced. The circulation is stimulated, and the tissue receives an optimal supply of oxygen. The active ingredients of the BIOMATRIX Masks are ideally absorbed. Thanks to the activation of the lymph flow, metabolic residues are removed. The skin immediately appears fresher and younger.

## BIOMATRIX CONCEPT - 4 easy steps to success

### Step 1

**Apply the BIOMATRIX Mask According to You Choice**  
The BIOMATRIX Masks are available in four different active substance combinations. The main ingredient is **collagen**. When applying these masks in combination with a micro massage via the IONTO-SKIN® REGULATOR, the active ingredients are optimally absorbed by the skin.

### Step 3

**Lay the Senso Membrane over the Biomatrix Mask**  
The Senso Membrane, Item No. 8228.031, is used in the Biomatrix treatment in order to increase the absorbing capacity of the skin for the nourishing ingredients.

### Step 2

**Moistening the Mask**  
The concentrated power active ingredients of the BIOMATRIX Masks are released when they come in contact with water. Use the BIOMATRIX Water pipette to moisten the mask, Item No. 8228.022.

### Step 4

**The Decisive Kick: The Micro Massage**  
With the IONTO SKIN REGULATOR  
Slide over your customer's skin with your hands and make the skin and muscles vibrate gently.



STARTER SET  
BIOMATRIX Concept



**Scope of delivery:**  
IONTO-SKIN REGULATOR (Item No. 8815.8228 ),  
BIOMATRIX-masks-set basic, 1 set of each one of the different masks,  
incl. sensomembrane (plastic film) and  
BIOMATRIX water pipette (Item No. 8228.023)

8409.58	BIOMATRIX Concept STARTER SET
---------	----------------------------------